

Anistel™

SWIFT

High-level disinfectant for surfaces



User guide

High-level disinfectant for surfaces

SWIFT is a highly effective chlorine dioxide solution designed for the rapid high-level disinfection of large hard surfaces within animal healthcare environments. SWIFT is an excellent preventative method for outbreaks.

Applications and uses

For rapid disinfection of large hard surfaces within animal healthcare environments, including:

- Operating theatres
- Isolation rooms
- Reception areas
- Consultation rooms
- Unoccupied animal habitats

Contents

- 50ml Base solution (5% solution of citric acid in demineralised water)
- 50ml Activator solution (2.1% sodium chlorite solution in demineralised water)

This makes 100ml chlorine dioxide in aqueous solution for dilution with water.

Kills organisms in five minutes in accordance with BS EN 14885

SWIFT is sporicidal, mycobactericidal, virucidal, fungicidal and bactericidal with a contact time of only five minutes.

- *Clostridium difficile* spores
- *Bacillus subtilis* spores
- MRSA
- Canine parvovirus
- Feline calicivirus
- *Candida albicans*
- Adenovirus
- *Pseudomonas aeruginosa*
- *Staphylococcus aureus*
- *Escherichia coli*
- *Proteus vulgaris*
- *Aspergillus brasiliensis* (formerly *niger*)
- *Mycobacterium avium*

Approved under DEFRA for Foot-and-mouth disease virus and Swine vesicular disease virus.

Health and safety

- Make up solution in a well ventilated area.
- Wear gloves and apron before commencing and wash hands after removing gloves and apron.
- Avoid contact with skin and eyes. Contact with the disinfectant may cause mild irritation. Wash affected areas with plenty of soap and water.
- In case of contact with eyes, rinse immediately with water and seek medical advice.
- For professional use only.

Storage

- Store product out of direct sunlight.
- Keep between 10°C and 35°C.

Disposal of packaging

- Dispose of empty packaging in accordance with local policy and national regulations.


Caution

- Do not use on soft furnishings or fabrics.
- Keep out of reach of children.
- TANK should be placed on a non-slip surface and remain stationary when in use.
- Do not place objects on top of TANK when in use.
- Rinse and dry TANK between uses.




SWIFT utilises a patented chlorine dioxide chemistry, a well-documented, highly effective biocide

How to use SWIFT

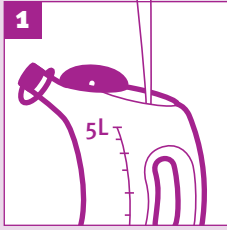

Pour five litres of ambient water into container.


Note: Do not use hot water.

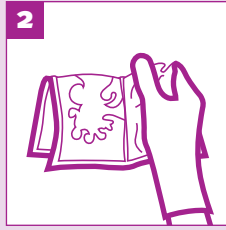


Take the SWIFT sachet and fold in half. Squeeze one side to burst contents through centre seal. Contents will start to turn yellow.




Allow a two minute dwell time.

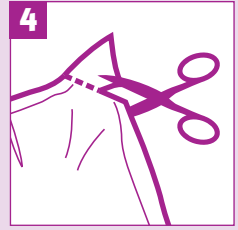

Product is ready to use. Tear or cut sachet.

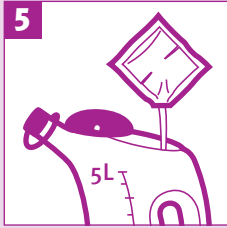


Pour sachet contents into the five litres of water.

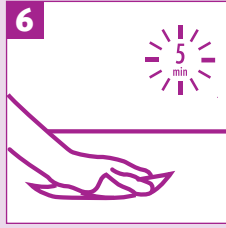

Apply SWIFT to surfaces with a mop for floors or a cloth for damp dusting. Leave to dry naturally to ensure a five minute contact time.

How to use SWIFT with TANK

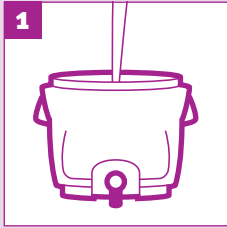

Pour five litres of ambient water into TANK.


Note: Do not use hot water.




Take two SWIFT sachets and fold each one in half. Squeeze one side to burst contents through centre seal. Contents will start to turn yellow.



Allow a two minute dwell time.

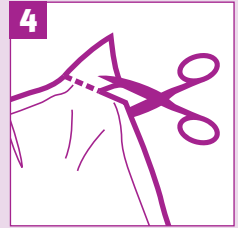


Product is ready to use. Tear or cut sachets.

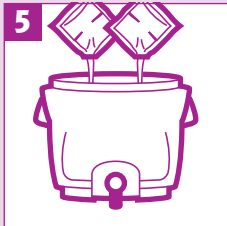


Pour sachet contents into the five litres of water in TANK. Then fill TANK with more water until the 10 litre marking is reached.



Screw close the TANK lid, ensuring it is fully tightened. Once mixed, solution can be kept for up to five days.

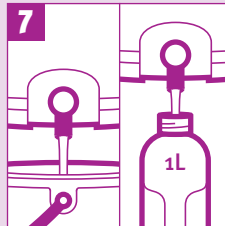

Use the tap on the front of TANK to decant SWIFT working solution into a bucket or into 1L dosing bottles.

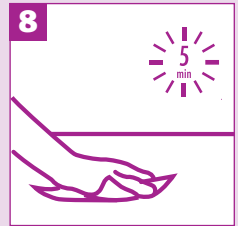

Apply SWIFT to surfaces with a mop for floors or a cloth for damp dusting. Leave to dry naturally to ensure a five minute contact time.



Manufactured in the United Kingdom by: Tristel Solutions Limited,
Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

New Zealand: Tristel New Zealand Limited, 23 Birch Avenue, Judea, Tauranga P.O. Box 3110
T +64 (0)7 5771560 - F +64 (0)7 5771567 - E info@tristel.co.nz

For Tristel patent information please visit: <http://www.our-patents.info/tristel>



Distributed by:

Ethical Agents Veterinary Marketing Limited, 54 Hobill Avenue,
Wiri, Auckland 2104, New Zealand

T 0800 800 624 - E info@ethicalagents.co.nz - W www.ethicalagents.co.nz

