

Tristel Trio™ Wipes System

A complete solution for high-level disinfection of non-lumened invasive medical devices.

 Refer to the User Guide for full instructions.

Notes:

- Always follow hospital protocols and device manufacturers' instructions.
- Disinfect hands and wear appropriate PPE when handling disinfectants and medical devices.
- An initial pre-clean should be performed immediately after medical device use as per hospital protocol.
- Take care not to rub or bend the instrument and pay particular attention to edges, ridges and indentations.
- Prepare for the decontamination process by taking one of each Wipe and the Activator Foam from the box.
- Do not use if packaging is damaged and/or if Wipes are dry.
- Use more Wipes if required to ensure the instrument is visibly clean.
- Keep the Wipe sachets to complete the Tristel Traceability book at the end of the decontamination procedure.
- Discard the Wipes and gloves in accordance with hospital guidelines.
- For professional use only.

FOR LARYNGOSCOPES 

STEP 1: CLEANING




Remove the Pre-Clean Wipe from its sachet and unfold it in the palm of your hand.

Wipe the Laryngoscope to remove all visible soil.

Discard the used Wipe and gloves to clinical waste.

Disinfect your hands and put on new gloves.

STEP 2a + 2b: ACTIVATING




Remove the Sporicidal Wipe from its sachet and unfold it in the palm of your hand.

Take the Activator Foam bottle and apply two doses of foam onto the Sporicidal Wipe.

Fold the Sporicidal Wipe in on itself and scrunch it during **15 seconds** to generate chlorine dioxide.

Ensure that the Wipe is completely covered in Foam.

STEP 2: HIGH-LEVEL DISINFECTING



Wipe the surface of the Laryngoscope in one movement to cover it with foam, ensuring all areas come into contact with the Wipe.

Discard the used Wipe to clinical waste.

Observe a minimum contact time of **30 seconds**.

STEP 3: RINSING



Remove the Rinse Wipe from its sachet and unfold it in the palm of your hand.

Wipe the Laryngoscope to remove excess foam.

Discard the used Wipe to clinical waste.

STEP 4: TRACEABILITY



Refer to the used Wipe sachets and Activator Foam bottle to complete the Tristel Quality Audit Trail Record Book.

Tristel

Created by: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

Australian sponsor: Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192
T 1300 680 898 - F +61 (0)3 9533 6193 - E mail-au@tristel.com

New Zealand: Tristel New Zealand Limited, P.O Box 15451, Tauranga, New Zealand
T +64 (0)7 5771560 - F +64 (0)7 5771567 - E info@tristel.co.nz

Hong Kong & Taiwan: Tristel Asia Limited, 21st Floor, 168 Electric Road, North Point, Hong Kong
T +852 2895 6968 - F +852 2985 6898

Copyright © Tristel Solutions
MK-Wal-041-5
January 2018

For Tristel patent information please visit: <http://www.our-patents.info/tristel>